CARDIAC ARREST

If the patient cannot be awakened and s/he does not breathe, DIAL THE EMERGENCY NUMBER 112. Start cardio-pulmonary resuscitation and ask others to bring you the defibrillator.


1. Switch on the FR2-defibrillator.



2. Take the pads out of pounch.



3. Place the pads on the patient`s clean and dry chest as directed.

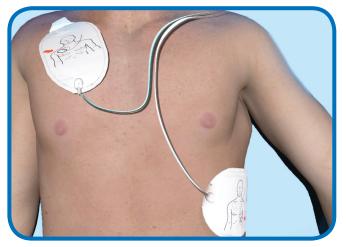

- **4.** The patient MUST NOT BE TOUCHED during the heart rhythm analysis.
- **5.** Press the blinking orange shock button, if the device instructs you to do so. DO NOT TOUCH THE PATIENT.



6. Continue the cardiopulmonary resuscitation immediately after you have given the shock. Follow the instructions given by the device.



You can stop the cardiopulmonary resuscitation when you are given a permission to do so, when the patient starts to breathe or when you are too tired to keep doing it.



Use the QR code on the left to upload CPR-D instructions to your mobile phone and watch the training video.

