## CARDIAC ARREST

If the patient cannot be awakened and s/he does not breathe, DIAL THE EMERGENCY NUMBER 112. Start cardiopulmonary resuscitation and ask others to bring you the defibrillator.


**1.** Switch on the FR3-defibrillator.



**2.** Take the pads out of pounch.



**3.** Place the pads on the patient`s clean and dry chest as directed.

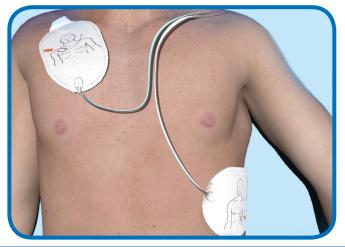

**4.** The patient MUST NOT BE TOUCHED during the heart rhythm analysis.

**5.** Press the blinking orange shock button, if the device instructs you to do so. DO NOT TOUCH THE PATIENT.



**6.** Continue the cardiopulmonary resuscitation immediately after you have given the shock. Follow the instructions given by the device.



You can stop the cardiopulmonary resuscitation when you are given a permission to do so, when the patient starts to breathe or when you are too tired to keep doing it.



Use the QR code on the left to upload CPR-D instructions to your mobile phone and watch the training video.



Tammed Oy • P.O.Box 298 • 33101 Tampere Finland • +358 3 2606 530 • www.tammed.fi